




PALTOP INTEGRATION IN 3shape

1. Open control panel

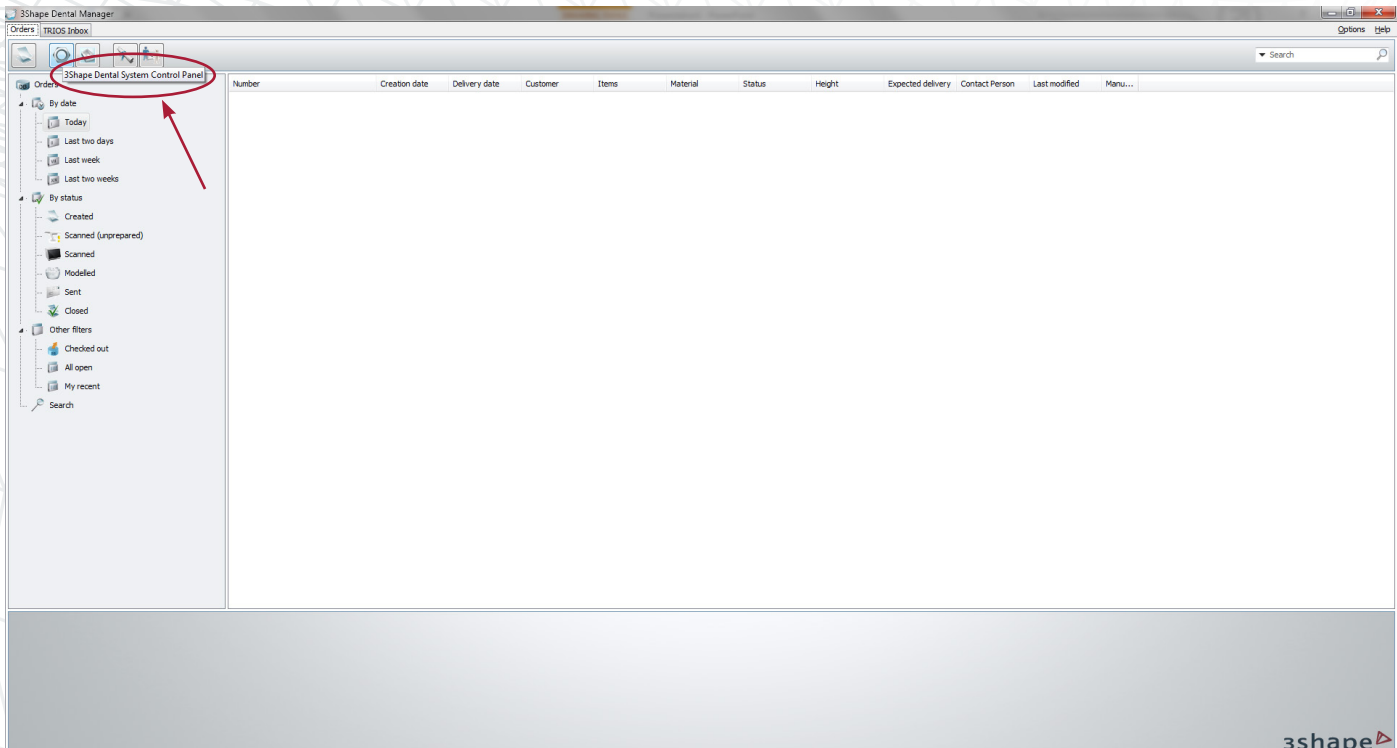

The screenshot shows the 3Shape Dental Manager software interface. The window title is "3Shape Dental Manager". The main area displays a table with columns: Number, Creation date, Delivery date, Customer, Items, Material, Status, Height, Expected delivery, Contact Person, Last modified, and Manu... The left sidebar contains a navigation menu with categories: "By date" (Today, Last two days, Last week, Last two weeks), "By status" (Created, Scanned (unprepared), Scanned, Modelled, Sent, Closed), and "Other filters" (Checked out, All open, My recent, Search). A red circle highlights the "3Shape Dental System Control Panel" icon in the top toolbar, with a red arrow pointing to it. The bottom right corner of the window features the "3shape" logo.



2. Choose: Import/Export



3. Click: Import materials



4. Select the Paltop DME file for installing the libraries