## Step-by-step instructions

### 1.0 Importing CAD libraries for prosthetic components into the exocad® CAD software

**Note:** CAD libraries for implant analogs see page 4

Previous versions of existing CAD libraries for prosthetic components (incl. from third party providers) should be deleted before importing the new library in order to minimize data errors.

#### Removing previous versions of the library

1. Choose the local hard disk (C:) within the Windows Explorer

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\* Exocad's installation folder is installed according to the following steps although the installation folder location may vary

3. Choose folder *exocad® DentalCAD* for the currently used version of the CAD

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7. Choose and delete folder with the currently used CAD library (in example *CAMLOG\_Titanium\_base\_CADCAM\_for\_exocad*)

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After removing previous versions of the library or libraries, the import of the current version (of the library) can be started.

1. Copy folder as part of the downloaded zipfile from the CAMLOG/iSy website


in folder *implant*.

(positioned in the same path as described for removing the former version of the library – see previous steps # 1 to 6)

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**Important:**  
Do not import a Zip archive – only unzipped folders can be used!

2. Double-check that the proper library has been imported in the *implant* folder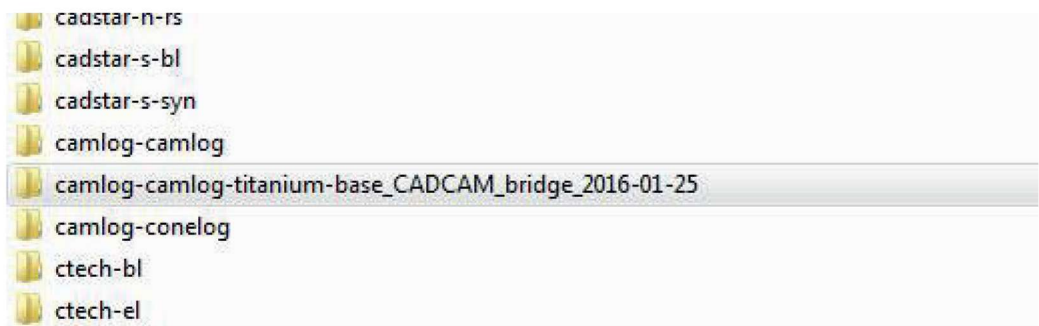

3. Close folder.


The exocad® CAD software can now be used with the current CAMLOG library

If you have another folder hierarchy please contact your exocad® reseller.

## Step-by-step instructions

### 2.0 Importing CAD Implant analog libraries into the exocad® CAD software


Disclaimer: Modelcreator is available in exocad® software version EB5164 and above. In combination with Implant analogs the following message appears (status March 2017):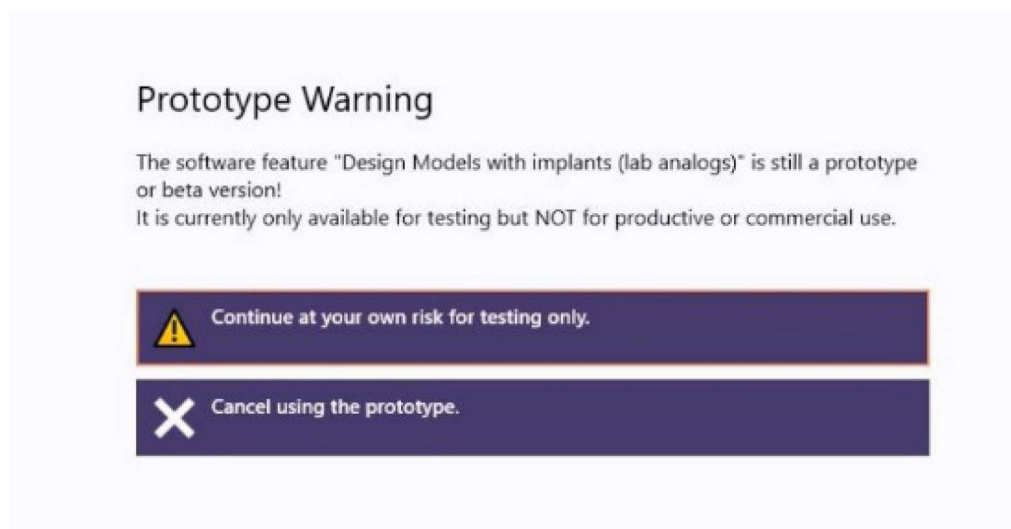

To import Implant analog libraries, follow the steps as described in chapter 1.0, but be aware they are saved in a different folder.

Implant analog libraries must be saved at

**Exocad > DentalCAD > DentalCadApp > library > modelcreator > implants**

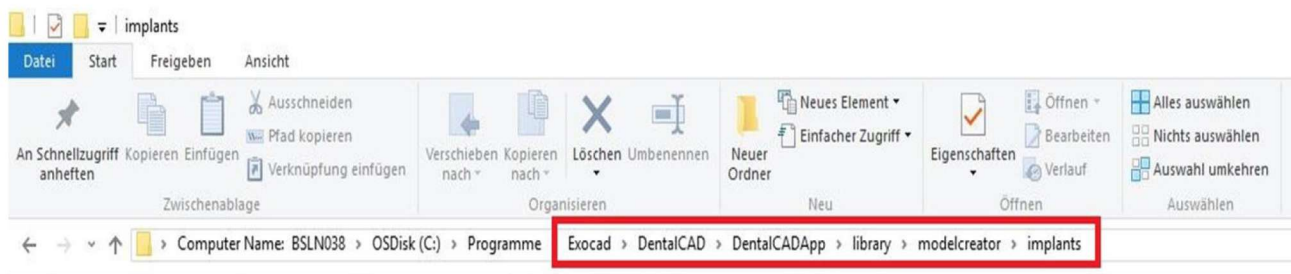If you have another folder hierarchy please contact your exocad® reseller.